

Usage Guidelines:

- Use supplied concentrated room temperature 10X buffer (RTB).
- Prepare master mix by combining the components in the following order: RTB, reagent grade water, internal standard in methanol, and IMCSzyme RT.
- Add the master mix to the urine sample and mix.
 - DO NOT VORTEX.**
- Incubate at room temperature 20-25 °C (do not apply heat).

Recommended Hydrolysis Reaction Setup

	Amount to add (μL)
10X RT Buffer	15
Water	135
ISTD in Methanol	10
IMCSzyme RT	5
Urine Sample	50

215 μL Total Volume

Content and Storage Guidelines:

Enzyme Solution:

Enzyme solution is stable at 2 - 8 °C for up to 24 months or expiration date on the bottle, in its original sealed package. The product is stable after opening for up to 12 months. **Do NOT freeze the enzyme solution.**


10X Room Temperature Buffer

RTB is sterile filtered and is to be used with the enzyme solution to obtain optimal pH. The original solution is stable at 2 - 30 °C up to 24 months in its sealed package. Master mix solution (buffer, water and enzyme) is stable for 4 weeks. Internal standards each have their own stability and have not been evaluated by IMCS.

Product Sterility:

IMCS products undergo strict quality control to ensure product sterility. The product can become contaminated if mishandled. Follow aseptic techniques to avoid contamination.

Enzyme Performance based on Recommended Ratios of Buffer, water, IMCSzyme RT and Urine:

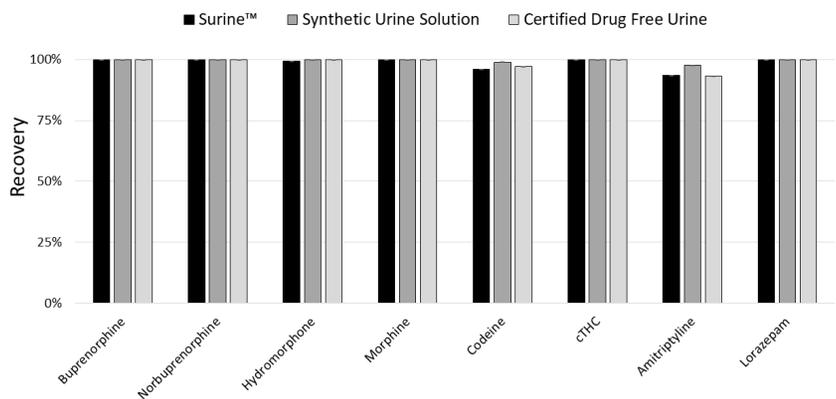

Enzyme Performance at pH 4-7:

Drug Analytes	Activity at pH 5.5 (U/mg)	pH							
		4	4.5	5	5.5	6	6.5	7	
Opiates	Morphine-3	9.6	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Hydromorphone	4.9	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Oxymorphone	3.6	Blue	Blue	Blue	Blue	Blue	Blue	Green
	Dihydrocodeine	4.3	Yellow	Blue	Blue	Blue	Blue	Blue	Yellow
	Codeine	4.1	Yellow	Blue	Blue	Blue	Blue	Blue	Yellow
Opioids	Tapentadol	13.3	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Norbuprenorphine	139.4	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Buprenorphine	40.8	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	O-desmethyiltramadol	5.3	Blue	Blue	Blue	Green	Green	Green	Green
Benzodiazepines	Oxazepam	34.9	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Lorazepam	180.7	Blue	Blue	Blue	Blue	Blue	Blue	Blue
	Temazepam	21.0	Blue	Blue	Blue	Blue	Blue	Blue	Blue
Anti-Depressant	Amitriptyline	5.8	Yellow	Yellow	Blue	Blue	Blue	Blue	Blue
Cannabinoids	cTHC	913.0	Blue	Blue	Blue	Blue	Blue	Blue	Blue

■ > 85% recovery
■ 75-85% recovery
■ < 75% recovery

Contact and Support:

For questions, concerns or technical support please contact us:

Email: custserv@imcstips.com

Toll-free Phone: +1 (888) 560-2073

Web: www.imcstips.com

Address: 110 Centrum Drive, Irmo, SC 29063 USA

Business Hours: Monday-Friday from 9 a.m. - 5 p.m. EST.

Certificate of Analysis and Safety Data Sheets are available under “resources” at: www.imcstips.com.

Product Warranty & Satisfaction Guarantee

IMCS warrants that this product conforms to the information contained in this publication at the time of the quality test date. The same efficient enzymatic hydrolysis result cannot be guaranteed without the use of the supplied buffer. **For R&D use only. Not for drug, household, or other use.**

This product is covered by one or more patents, pending patents, trademarks and/or copyrights owned or controlled by IMCS Inc.